making work easy

- DE
- EN
- FR
- IT
- ES
- PT
- TR
- RU
- UK
- ZH
- JA
- KO
- AR

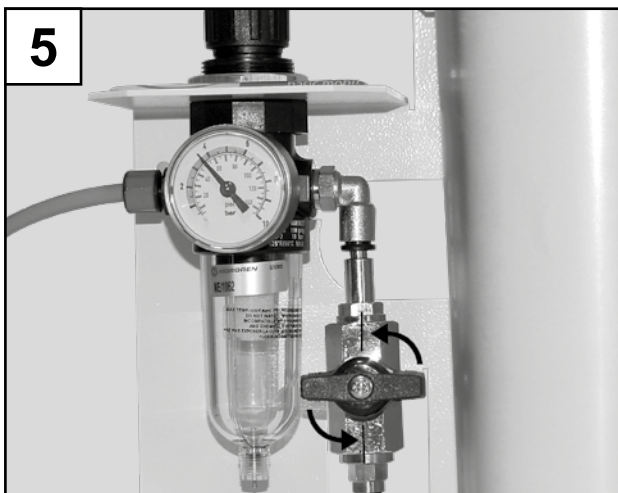
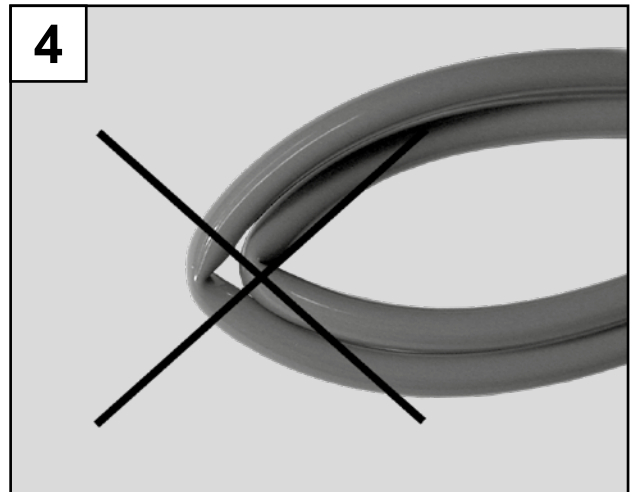
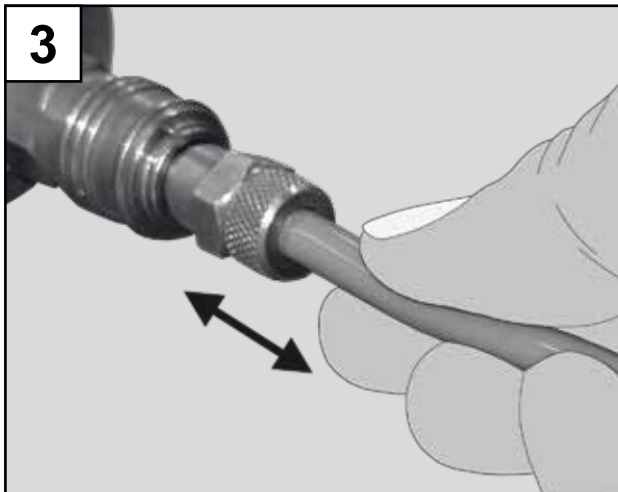
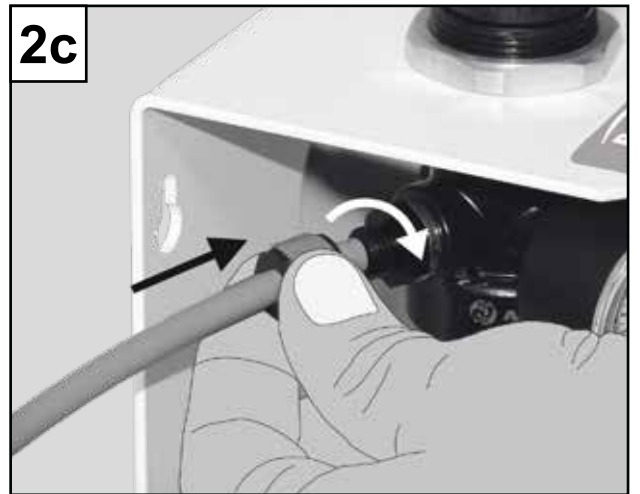
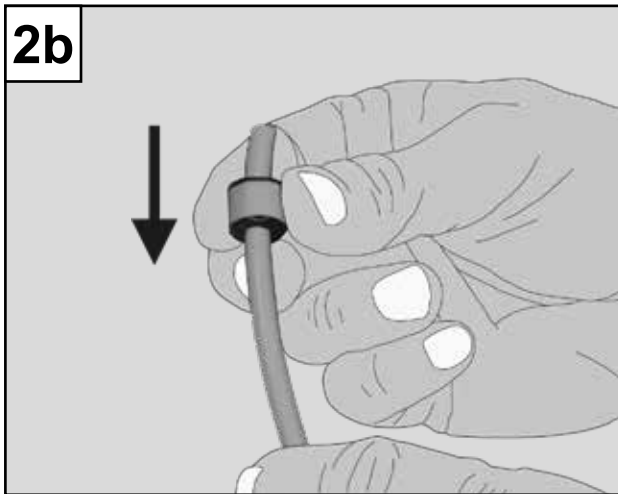
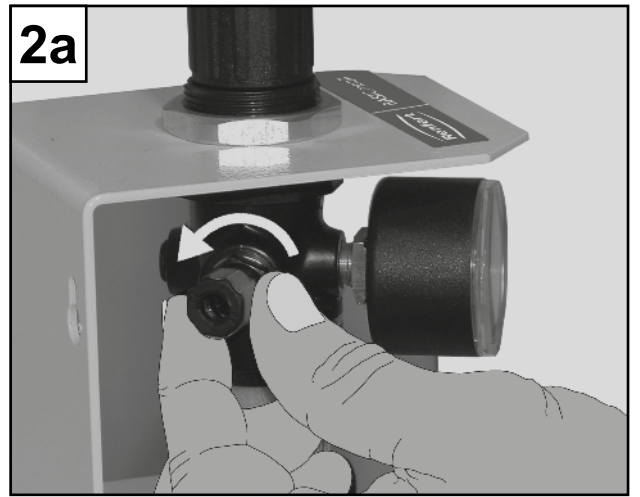
# Basic mobil

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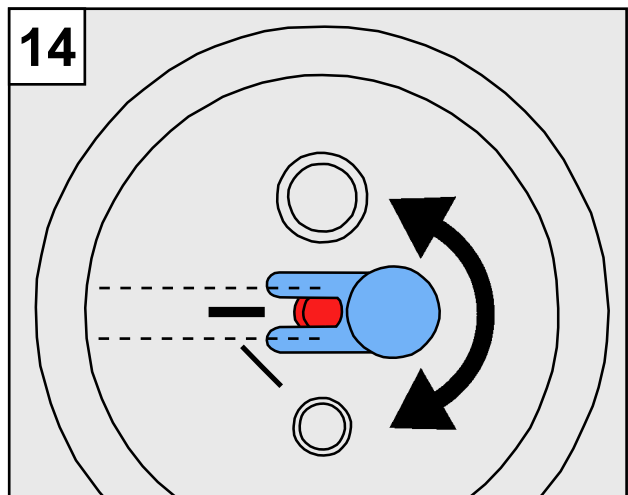
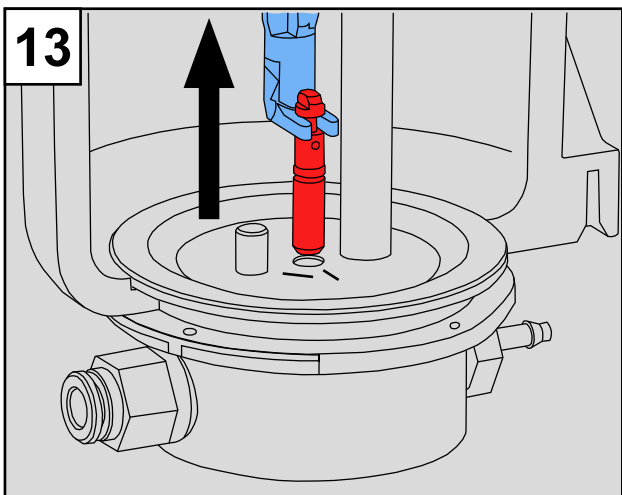
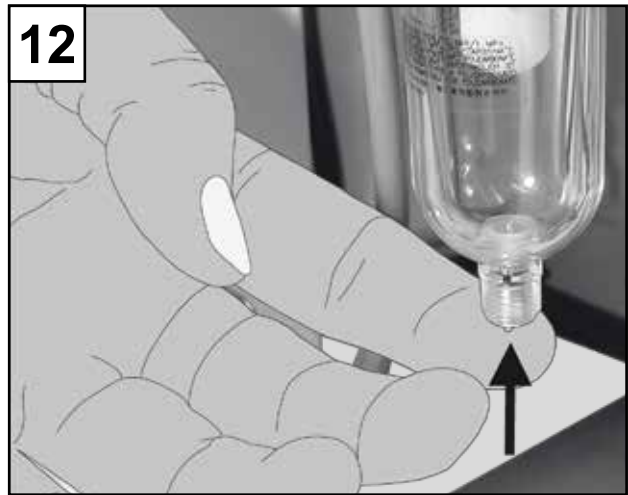
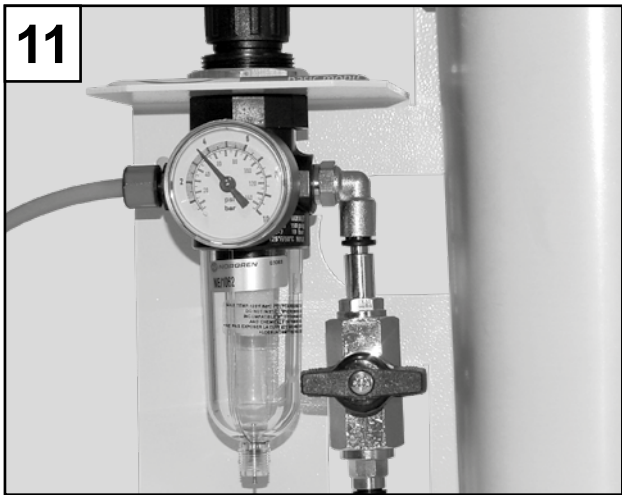
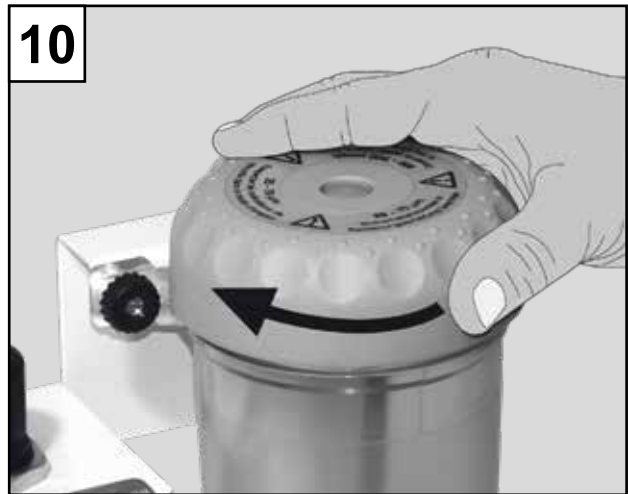
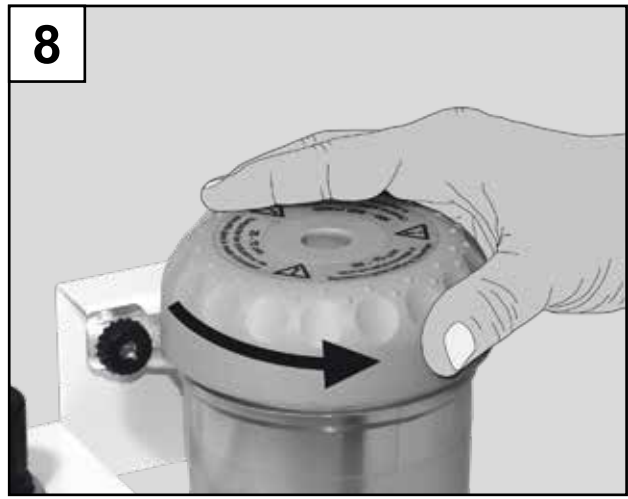
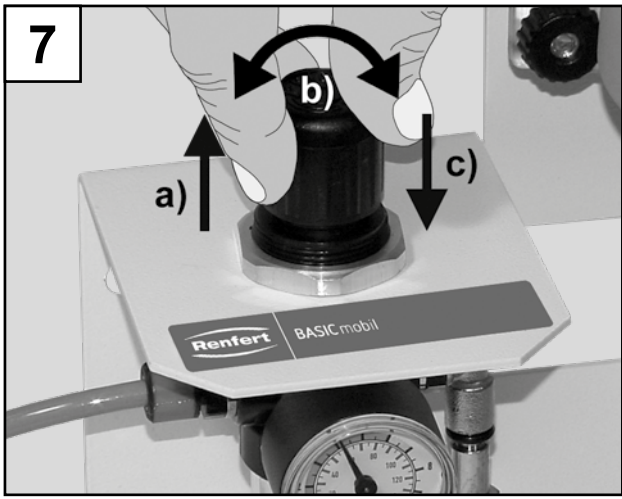
Made in Germany

21-9093 11102019

# Basic mobil

## Nr. 2914

DEUTSCH

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## 1. Einleitung

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ENGLISH

## 1. Introduction

We are pleased with your decision to purchase the *Basic mobil*. The mobile sandblasting unit is connected to the compressed air or to the recyclable blaster *Vario-Jet* resp. *Vario basic*.

## 2. Application Area

The *Basic* line of sandblasting units are designed for use in dental laboratories to remove investment residues, oxides on cast objects, and for surface treatment.

## 3. Safety Information

- Always disconnect the unit from the compressed air and power sources prior to beginning any maintenance work.
- When working on the tank (filling up, cleaning, maintenance) wear protective glasses for protecting your eyes.
- Any residue of abrasive material on the gasket may lead to leakage and early wear of the gasket. After filling up clean the thread and the gasket and close the lid properly.
- Solvents and tensides can create micro-cracking in the plastic (danger of explosion!). Clean the tank and the lid only with a dry cloth. Do not write or stick something on the tanks.
- Check the tanks and the lids regularly on damages and replace them in case of doubt.
- Check the tank cover for correct seating prior to beginning operation. Covers that are not securely closed can suddenly be blown off as the tank pressure increases. The resulting flying parts and abrasive represent a serious hazard.
- **Never** operate the blasting units without suitable dust extraction and appropriate protective gear, as this could result in **health hazards**. The type of dust extraction should be selected on the basis of the dust being generated. Follow here **absolutely** EN 60335-2-69 appendix AA or ask the appropriate authorities.
- Improper use can result in the risk of eye or skin injuries.
- Never direct the blasting material towards eyes or unprotected areas of skin!

**Caution:** Failure to wear proper eye protection can result in eye injury due to particles suspended in the air. Always wear proper eye protection to protect your vision!

- Do not employ the foot switch when only the compressed air supply hose is connected. The loose hose could wrap around itself and cause serious injury.
- Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.
- It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical & Electronic Technology).
- Information on REACH and SVHC is available on our website [www.renfert.com](http://www.renfert.com), in the Support area.

### 3.1 Liability Exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- The product is employed for any purposes other than those cited in the operating instructions;
- The product is altered in any way other than those alterations described in the operating instructions;
- The product is repaired by other than an authorized facility or if any but Renfert OEM parts are employed;
- The product continues to be employed, despite obvious safety faults;
- The product is subjected to mechanical jarring or is dropped.

## 4. Installation / Commissioning

### 4.1 Setting up and Connecting the Unit with Foot Switch

1. Inspect the tank for proper installation (Fig. 1).
2. Connect the foot switch / sandblasting unit (red) (Fig. 2a, 2b, 2c).
3. Connect the foot switch / air supply (P) (Fig. 3).
4. Do not bend or kink the hose (Fig. 4)!

### 4.2 Setting up and Connecting the Unit without Foot Switch

1. Inspect the tank for proper installation (Fig. 1).
2. Connect the compressed air hose / sandblasting unit (red) (Fig. 2a, 2b, 2c).
3. Connect the compressed air hose / air supply (P) (Fig. 3).
4. Do not bend or kink the hoses (Fig. 4)!

### 4.3 Adjusting the Operating Pressure

1. Put in shut off-valve in a vertical position (Fig. 5).
2. Set the blasting pressure from 1 - 6 bar [14.5 - 87 psi] (Fig. 7) and in case you use a foot-switch, it has to be pressed additionally (Fig. 6).

### 4.4 Filling the Sandblasting Tank

 **Close the shut-off valve before filling the tanks (Fig. 11). Before closing the tank, clean the threads in the cover and on the tank, in order to remove any abrasive residue.**

1. Unscrew the tank cover (Fig. 8).
2. Add abrasive up to the maximum fill level (the point where the tank jacket thickens just below the thread) (Fig. 9a + 9b). Make sure the abrasive being used is compatible with the sandblasting tank.
3. Reinstall the tank cover (Fig. 10).

 **Do not cross-thread the cover.**

Always use only clean, dry abrasive of the appropriate grain size (refer to the accessories).

**The sandblasting unit is now operational.**

## 5. Cleaning / Maintenance

 **Use only solvent-free cleansers (e.g., soapy water). Wipe out the tanks and lids only with a dry cloth!**

### 5.1 Condensation

1. Inspect the water separator.
2. Close the shut off-valve (Fig. 11).
3. Drain any water off through the valve (Fig. 12), employing the foot switch (Fig. 6).

### 5.2 Blasting Nozzle Replacement

1. Refer to insert no. 21-9162 for installation.
2. Inspect for abrasive residue.
3. Inspect the seal ring position.

### 5.3 Sandblasting Hose Replacement

1. Disconnect the foot switch from the air supply (Fig. 3).
2. Refer to insert no. 21-9162 for the installation instructions.

### 5.4 Cleaning the dosing nozzle

The dosing nozzle from the mixing chamber can become blocked due to contamination and moist blasting sand.

For cleaning, the dosing nozzle is removed using the supplied tool and replaced after cleaning.

- Vacuum clean the blasting tank completely.
- Hold the dosing nozzle with the tool and pull upwards (Fig. 13).
- Clean the dosing nozzle (e.g. blow with compressed air).
- Replace dosing nozzle.

 **Before remounting, the dosing nozzle should be completely dry!**

- The dosing nozzle must be adjusted with blasting tanks of 25 - 70 µm (Fig. 14)!

 **see also supplement „Cleaning the dosing nozzle“**

## 6. Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at [www.renfert.com/p918](http://www.renfert.com/p918).

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

## 7. Warranty

Provided the unit is properly used, Renfert warrants all parts of the *Basic mobil* for a **period of 3 years**. In case of any claims for warranty original dealers' invoice is required. Parts which are subject to natural wear and tear (wear parts) and consumables are excluded from the guarantee. These parts are marked in the spare part list.

The warranty is voided in case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or; in case of unusual influences or influences not in compliance with the utilization instructions. Warranty service shall not extend the original warranty.

## 8. Technical data

Working pressure:	1 - 6 bar [14.5 - 87 psi]
Connecting pressure, external:	6 - 8 bar [87 - 116 psi]
Air consumption:	120 l/min at 6 bar [4.24 cfm at 87 psi]
Ø Compressed air hose:	
- interior:	4 mm [157 mil]
- exterior:	6 mm [236 mil]
Dimensions (W x H x D):	220 x 260 x 130 mm [8.7 x 10.2 x 5.1 inch]
Tank capacity:	1000 ml [34 fl oz]
Weight (empty), approx.:	2.5 kg [5.5 lbs]

## 9. Notes

Flat blasting with sheet dies:

Art. No. 90002-1207	0.3 x 3.5 mm [0.012 x 0.138 inch]
Art. No. 90003-1739	1.5 x 3.5 mm [0.059 x 0.138 inch]

When blasting glass ceramics, refer to manufacturer's notes.

## 10. Standard Delivery

- 1 *Basic mobil*
- 1 Foot switch (only with no. 2914-3050 / 2914-3250)
- 1 Connection set
- 1 Set of Operating Instructions, with attachment
- 1 Sample of blasting material

## 11. Delivery Versions

- No. 2914-3050 *Basic mobil* 25 - 70 µm, with foot switch
- No. 2914-3250 *Basic mobil* 70 - 250 µm, with foot switch

## 12. Accessories

### Abrasives

#### Cobra aluminium oxide

- No. 1594-1105 25 µm [500 mesh], white 5 kg canister
- No. 1594-1205 50 µm [270 mesh], white 5 kg canister
- No. 1594-2220 50 µm [270 mesh], white 20 kg bucket
- No. 1584-1005 90 µm [170 mesh], white 5 kg canister
- No. 1583-1005 110 µm [150 mesh], white 5 kg canister
- No. 1583-1020 110 µm [150 mesh], white 20 kg bucket
- No. 1587-1005 125 µm [115 mesh], pink 5 kg canister
- No. 1587-1020 125 µm [115 mesh], pink 20 kg bucket
- No. 1585-1005 250 µm [60 mesh], white 5 kg canister
- No. 1585-1020 250 µm [60 mesh], white 20 kg bucket

#### Rolloblast glass beads

- No. 1594-1305 50 µm [400 - 200 mesh] 5 kg canister
- No. 1594-2312 50 µm [400 - 200 mesh] 12.5 kg canister
- No. 1589-1005 100 µm [170 - 100 mesh] 5 kg canister

# 13. Troubleshooting Guide

Problem	Possible cause	Corrective action
<b>No air and no abrasive.</b>	<ul style="list-style-type: none"> <li>Blasting nozzle blocked at the hand-piece.</li> <li>Foot switch connecting lines kinked.</li> </ul>	<ul style="list-style-type: none"> <li>Unscrew the nozzle from the hand-piece and blow it out.</li> <li>With the nozzle removed, blow out the blasting hose by activating the foot pedal.</li> <li>Inspect/correct the connection lines.</li> </ul>
<b>Only air, no or very little abrasive.</b>	<ul style="list-style-type: none"> <li>The dosage nozzle in the mixing chamber is blocked due to contaminated or moist sand.</li> <li>Mixing chamber filter combination blocked.</li> <li>Abrasive contains impurities or is moist.</li> </ul>	<ul style="list-style-type: none"> <li>Cleaning the dosing nozzle, see chapter 5.4.</li> <li>Change filter cartridge (Wrench SW14).</li> <li>Replace the abrasive.</li> </ul>
<b>Unit blasts continuously.</b>	<ul style="list-style-type: none"> <li>Foot switch connected incorrectly.</li> <li>Valve in foot switch is defective.</li> </ul>	<ul style="list-style-type: none"> <li>Inspect / correct the foot switch connection.</li> <li>Replace foot switch.</li> </ul>
<b>Compressed air leaking from the foot switch.</b>	<ul style="list-style-type: none"> <li>Pneumatic hoses reversed.</li> </ul>	<ul style="list-style-type: none"> <li>Reverse the connections.</li> </ul>
<b>Foot switch fails to switch on.</b>	<ul style="list-style-type: none"> <li>No pressure at the compressed air connection.</li> <li>Pneumatic hoses are kinked.</li> </ul>	<ul style="list-style-type: none"> <li>Inspect the compressed air lines and/or the compressor.</li> <li>Reposition the hoses.</li> </ul>

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**Renfert GmbH, Industriegebiet, 78247 Hilzingen / Germany**
Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany